

**REF** BTLE-1250

RM/N ES01  
Rev. 112923

STERILE R

*Bridge  
to  
Life*

**EasiSlush®**

## Sodium Chloride Solution for Sterile Slush Preparation

0.9% Sodium Chloride Irrigation, USP - 1250mL

**STERILE \* NONPYROGENIC  
NOT FOR INJECTION \* SINGLE DOSE CONTAINER**

See Accompanying Instructions For Use

**For Use Only In Slush Preparation to Establish and  
Maintain Hypothermia in Donor Organs for Transplant.**

**WARNING: Not to Induce Regional Hypothermia in Living Patients**

Do not remove from clear, outer pouch until ready for use. The sterile-barrier, inner pouch maintains sterility of the outer surface of the slush container.



Store at 2°C to 25°C (36°F to 77°F). Avoid excessive heat.

Avoid freezing except during controlled slush preparation – see instructions.

After removing the clear, outer pouch, check that all seals on the sterile-barrier, inner pouch are intact. If any broken seals, holes or leaks are detected, discard unit as sterility may be impaired.

Do NOT use this solution if obvious particulate matter, precipitates or contamination are evident in the solution.

**CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

Manufactured for:  
Bridge to Life Ltd.  
707 Skokie Boulevard, Suite 340  
Northbrook, IL 60062 USA